


Paediatric SELECTION CHART

Durham and Darlington NHS Foundation Trust

Specimens must be labelled with a MiniCollect® label (KFK218).

Tubes 3, 4 and 5: Label must be placed on the inner MiniCollect® tube (not the separate carrier tube).

Tubes 1, 2 and 6: Carrier tube is pre-attached so labelling of carrier tube is permitted.

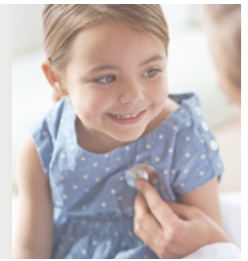


MiniCollect®

The list of tubes follow the correct Order of Draw

	Volume & Ordering Code	MiniCollect Cap Colour	Tube Type	Tests	Special Instructions
1	0.25 - 0.5ml KFK816	Lavender	MiniCollect Complete EDTA K3	Transfusion Specimens Labels must be: <ul style="list-style-type: none"> • Handwritten (addressograph labels are not accepted) • Signed • Dated • Timed • Labelled at the patient's bedside 	All samples for Transfusion must be handwritten onto the MiniCollect® Label and attached to the tube. Labels should be completed according to the Blood Transfusion labelling policy (POL/Transfusion/0012). This a zero tolerance policy and no amendments to the label can be made once it is received in the laboratory. MiniCollect Complete has a pre-attached carrier tube which CANNOT be removed. Placing label on the carrier tube is permitted on MiniCollect Complete tubes ONLY
2	0.25 - 0.5ml KFK816	Lavender	MiniCollect Complete EDTA K3	Haematology, FBC, etc.	MiniCollect Complete has a pre-attached carrier tube which CANNOT be removed. Placing label on the carrier tube is permitted on MiniCollect Complete tubes ONLY
3	1ml KFK833	Light Blue	Trisodium Citrate Solution	Coagulation, D-dimer	
4	0.8ml KFK829	Light Green	Lithium Heparin Gel	Chromosomes and special tests needing plasma	
5	0.25ml KFK830	Grey	Fluoride Oxalate	Glucose, Lactate, Alcohol	
6	0.5 - 0.8ml KFK813	Gold	MiniCollect Complete Serum with Separation Gel	All Biochemistry	MiniCollect Complete has a pre-attached carrier tube which CANNOT be removed. Placing label on the carrier tube is permitted on MiniCollect Complete tubes ONLY

CORRECT LABELLING: TUBES 3, 4 and 5

The label MUST be placed onto the MiniCollect® tube overlapping at the bottom. (Not the separate carrier tube)

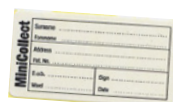

MiniCollect® Label
KFK218



MiniCollect® Carrier Tube
13 x 75mm
KFK837

greiner
BIO-ONE